

CS PLUS 10 ECO



dati CS PLUS 10

DESCRIPTION

Disposable container for sharp and cutting objects. Item designed for frequent use (Hospitals, Healthcare Centres, Clinics)

GENERAL DATA

- ▶ Material: Plastic: 60% of recycled polypropylene
- ▶ Colour: Yellow body in order to signal the dangerous content, white lid. Materials coloured with masterbatch without heavy metals and cadmium. Materials compliant to REACH
- ▶ Shape: Square base
- ▶ Weight: 570 g
- ▶ Max Load: 4 kg
- ▶ Nominal capacity: 10,00 L
- ▶ Real capacity: 9,10 L
- ▶ Useful capacity: 7,76 L
- ▶ External dim.: 21,5 x 21,5 x 32 H cm (lid assembled)



LOGISTIC DATA

- ▶ Pcs per carton: 33 (body and lid not assembled)
- ▶ Carton dimensions: 79 x 53 x 49 H cm
- ▶ Pcs per pallet: 264 pz
- ▶ Pallet dimensions: 120x80x231 H cm

CERTIFICATIONS:

- ▶ Compliant to NF mark



- ▶ Compliant to ISO 23907-1:2019 Kite Mark



- ▶ Compliant to ADR/UN REC



INSTRUCTIONS

Lid assembly:

Place the opening of the lid above the label and press firmly on the opposite corners clicking all the joints and after on the opposite sides.



Temporary closure:

Lower the lid holding the tongue on «1PROV», then push the tongue to block the edge. At this stage closure is temporary



Permanent closure:

Place the lid in temporary closure and press firmly on the two central «1 DEF» and after on the «2 DEF». At this stage, closure is permanent.



Needles removal:

Use the «V» removal device to separate needles and syringes



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
Use the «marguerite» device to unscrew the insulin pen



INSTRUCTIONS

Holders:

Wall/trolley
bracket with
grabs



USE:

- ▶ During use, close the cap in temporary closure position
- ▶ Close permanently once full
- ▶ Large opening to ease the disposal of scalpels, test tubes, etc
- ▶ Suitable to autoclave at 134°C for 18 minutes
- ▶ Once filled, the containers have to be disposed through incineration, or following national regulations

SAFETY INFORMATION:

- ▶ Use the product only with the appropriate lid
- ▶ Do not use the product before fitting the lid
- ▶ Do not attempt to rescue objects already in the container
- ▶ Respect the filling level indicated: max. 85% of real capacity
- ▶ Do not try to open the container once permanently and irreversibly sealed
- ▶ Container to dispose according to the existing law
- ▶ Follow the instructions of use to ensure the safety of the container
- ▶ Use handles to move the container

ADVANTAGES:

- ▶ Lid provided with safe closure device, permanent closure
- ▶ Easy to use and to handle
- ▶ Sharps and cutting objects resistants
- ▶ Handles available
- ▶ Lid provided with needles removal devices
- ▶ Suitable to autoclave
- ▶ Maximum filling level indicated
- ▶ Several holders available
- ▶ Irreversible permanent closure