

## Technical Sheet

Rev. 00 - 13/04/2016 - Mod. 73A - EN

# CS 10 PLUS



### GENERAL DATA

**Disposable container for sharp and cutting objects. Item designed for frequent use (Hospitals, Healthcare Centres, Clinics)**

- ▶ **Material:** Plastic material 100 % virgin Polypropylene
- ▶ **Colour:** Yellow body and red lid in order to signal the dangerous content. Materials coloured with masterbatch without heavy metals and cadmium. Materials compliant to REACH
- ▶ **Thickness:** Bottom and sides: 2,0 mm - Lid and closure: 2,0 mm
- ▶ **Shape:** Square
- ▶ **Weight:** 600 gr.
- ▶ **Max Load:** 3400 gr.
- ▶ **Nominal capacity:** 10,00 Lt.
- ▶ **External dimensions:** 221x221x320 H mm (lid assembled)



### LOGISTIC DATA:

- ▶ **Pcs per carton:** 33 (body and lid not assembled)
- ▶ **Carton dimensions:** 79x60x48 H cm
- ▶ **Pcs per pallet:** 264 pcs.
- ▶ **Pallet dimensions:** 120x80x204 h cm


### CERTIFICATIONS:

- ▶ Compliant to AFNOR NF X-30-500
- ▶ NF Mark
- ▶ Compliant to ADR - UN approved
- ▶ Kite Mark - Compliant to ISO 23907

## INSTRUCTIONS

### ► Lid Assembly:

Place the opening of the lid above the label and press firmly on the opposite corners clicking all the joints and after on the opposite sides.

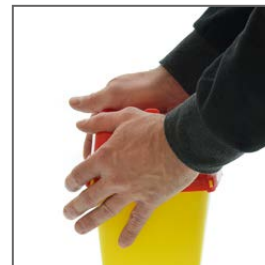

### ► Temporary closure:

Lower the lid holding the tongue on «1PROV», then push the tongue to block the edge

► At this stage closure is temporary



### ► Permanent closure:

Place the lid in temporary closure and press firmly on the four «2DEF» in succession from one side to the other. At this stage, closure is permanent



### ► Needles removal:

Use the "V" removal device to separate needles and syringes

Use the "marguerite" device to unscrew the insulin pen



► **Holders:**

Wall trolley bracket with grabs



**SAFETY INFORMATION:**

- Use the product only with the appropriate lid
- Do not use the product before fitting the lid
- Do not attempt to rescue objects already in the container
- Respect the filling level indicated: 80% of real capacity
- Do not try to open the container once permanently and irreversibly sealed
- Container to dispose according to the existing law
- Follow the instructions of use to ensure the safety of the container
- Use handles to move the container

**ADVANTAGES:**

- Lid provided with safe closure device, permanent closure, prevents from waste reflux
- Easy to use and to handle
- Sharp and v objects resistant
- Handles available
- Lid provided with needles removal devices
- Suitable to autoclave (compliant to NFX 30-500 even after sterilization)
- Maximum filling level indicated
- Several holders available
- Irreversible permanent closure
- To be disposed through incineration, releasing no toxic emissions