

Technical sheet

Rev. 01 of 05/02/2020 - Mod. 73-J1-EN

DAILY 7




GENERAL DATA

Disposable container for sharp and cutting objects disposal

Item designed for frequent use (Hospitals, Healthcare Centres, Clinics)

- ▶ **Material:** Plastic material 100 % virgin Polypropylene
- ▶ **Colour:** Yellow body and red lid in order to signal the dangerous content
Materials coloured with masterbatch without heavy metals and cadmium
Materials compliant to REACH
- ▶ **Shape:** Square base
- ▶ **Weight:** 360 gr.
- ▶ **Max load:** 3.200 gr.
- ▶ **Nominal capacity:** 7,00 Lt.
- ▶ **Real capacity:** 6,60 Lt.
- ▶ **Useful capacity:** 5,60 Lt.
- ▶ **External Dimensions:** 190 x 190 x 320 mm H (lid assembled)
- ▶ **Base dimensions :** 137 x 137 mm

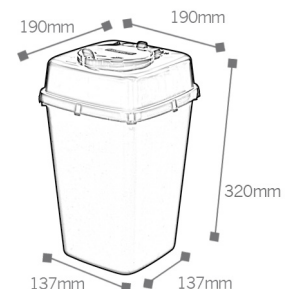

LOGISTIC DATA:

- ▶ **Pcs per carton:** 30 (body and lid not assembled)
- ▶ **Carton dimensions:** 60 x 40 x 36,5 cm
- ▶ **Pcs per pallet:** 720 pcs.
- ▶ **Pallet dimensions** 120 x 80 x 231 cm

CERTIFICATIONS:

- ▶ Compliant to ADR - UN Certificate
- ▶ Compliant to ISO 213907-1 - Kite mark
- ▶ Compliant to TRBA 250

INSTRUCTIONS

► Lid assembly:

Place the opening of the lid above the label and press firmly on the opposite corners, then the opposite sides.



► Temporary closure:

Lower the lid holding the tongue on "PROV", then push the tongue to block the edge

► At this stage, closure is temporary



► Permanent closure:

Place the lid in temporary closure and press firmly on th 2 "DEF". At this stage, closure is permanent



► Needles removal:

Needles:
Use the "V" removal device to separate needle and syringe and/or to remove the drawing needle

Insulin pens:
Use the "marguerite" device to unscrew the insulin pen.

Opening for waste insertion:
large opening in order to ease the insertion of needles, scalpels, test-tubes, etc



▶ **Available holders:**

Wall/trolley bracket with grabs

SAFETY INFORMATION:

- ▶ Use the product only with the provided lid
- ▶ Do not use the product before fitting the lid
- ▶ Do not attempt to rescue objects already in the container
- ▶ Do not exceed the filling level indicated on the lid
- ▶ Do not try to open the container once permanently and irreversibly sealed
- ▶ Respect the filling level indicated: 80% of real capacity
- ▶ Container to dispose according to the existing law
- ▶ Containers are tested for not being perforated by sharp objects allowing safety to the operators, as long as the container is used properly according to instructions
- ▶ Use handle to move the container

ADVANTAGES:

- ▶ Simple and practical spring-closure device
- ▶ Easy to handle temporary closure, to use with one hand
- ▶ Irreversible permanent closure
- ▶ Maximum filling level indicated
- ▶ Suitable to autoclave at 134° for 18 min
- ▶ Handles available to ease transport
- ▶ To be disposed through incineration, releasing no toxic emissions
- ▶ Resistant to perforation of sharps and cutting objects normally used
- ▶ Large opening to ease the insertion of voluminous objects