

## Technical Sheet

Rev. 02 of 06/02/2020 - Mod. 73-A9\_EN


# CS PLUS 22



### GENERAL DATA

**Disposable container for sharp and cutting objects. Item designed for frequent use (Hospitals, Healthcare Centres, Clinics)**

- ▶ **Material:** Plastic material 100 % virgin Polypropylene
- ▶ **Colour:** Yellow body and red lid in order to signal the dangerous content. Materials coloured with masterbatch without heavy metals and cadmium. Materials compliant to REACH
- ▶ **Shape:** Rectangular base
- ▶ **Weight:** 1100 gr.
- ▶ **Max Load:** 5000 gr.
- ▶ **Nominal capacity:** 22,00 lt.
- ▶ **Real capacity:** 21,10 Lt.
- ▶ **Useful capacity:** 17,80 Lt
- ▶ **External dimensions:** 375x272x345 H mm (lid assembled)

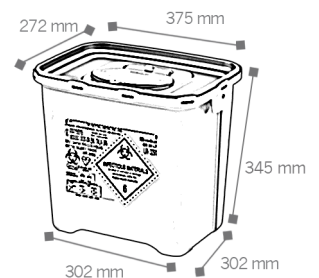

### LOGISTIC DATA:

- ▶ **Pcs per carton:** 10 (body and lid not assembled)
- ▶ **Carton dimensions:** 40x30x105 h cm
- ▶ **Pcs per pallet:** 160 pcs.
- ▶ **Pallet dimensions:** 120x80x235 h cm


### CERTIFICATIONS:

- ▶ Compliant to ADR -UN approved
- ▶ Compliant to ISO 23907-1 - Kite Mark
- ▶ Compliant to TRBA 250

## INSTRUCTIONS

### ► Lid Assembly:

Place the opening of the lid above the label and press firmly on the opposite corners clicking all the joints and after on the opposite sides.

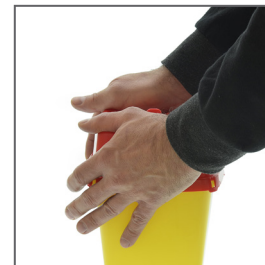

### ► Temporary closure:

Lower the lid holding the tongue on «1PROV», then push the tongue to block the edge

► At this stage closure is temporary



### ► Permanent closure:

Place the lid in temporary closure and press firmly on the two central «1 DEF» and after on the «2 DEF». At this stage, closure is permanent.



### ► Needles removal:

Use the «V» removal device to separate needles and syringes

Use the «marguerite» device to unscrew the insulin pen



▶ **Holders:**

Wall trolley bracket with grabs



**SAFETY INFORMATION:**

- ▶ Use the product only with the appropriate lid
- ▶ Do not use the product before fitting the lid
- ▶ Do not attempt to rescue objects already in the container
- ▶ Respect the filling level indicated: max. 85 % of real capacity
- ▶ Do not try to open the container once permanently and irreversibly sealed
- ▶ Container to dispose according to the existing law
- ▶ Follow the instructions of use to ensure the safety of the container
- ▶ Use handles to move the container

**ADVANTAGES:**

- ▶ Lid provided with safe closure device, permanent closure
- ▶ Easy to use and to handle
- ▶ Sharps and cutting objects resistant
- ▶ Handles available
- ▶ Lid provided with needles removal devices
- ▶ Suitable to autoclave
- ▶ Maximum filling level indicated
- ▶ Several holders available
- ▶ Irreversible permanent closure