

## Technical Sheet

Rev. 00 06/09/2022 - Mod. 72\_V2\_EN

# DISPO 0,5



### GENERAL DATA

**Disposable container for sharp and cutting objects disposal Item designed for low volumes of waste (care tray, trolley, domestic care, doctor's office)**

- ▶ **Closure:** Lid with porthole
- ▶ **Material:** Ecologic plastic material 100% virgin polypropylene, coloured with masterbatch without cadmium. Material compliant to REACH
- ▶ **Colour:** Yellow body and lid in order to signal the dangerous content
- ▶ **Shape:** Truncated cone
- ▶ **Weight:** 64 gr.
- ▶ **Max Load:** 250 gr.
- ▶ **Nominal Capacity:** 0,500 Lt.
- ▶ **External dimensions:** Ø 81 x 151 mm H (lid assembled)



### LOGISTIC DATA:

- ▶ **Pcs per carton:** 187 pz. (body and lid not assembled)
- ▶ **Carton dimensions:** 60x40x36 H cm
- ▶ **Pcs per pallet:** 4488
- ▶ **Pallet dimensions:** 120x80x228 H cm

### CERTIFICATIONS:

- ▶ Compliant to ADR - UN Certificate
- ▶ Compliant to ISO 23907-1:2019 Kite Mark
- ▶ TRBA 250

## INSTRUCTIONS

### ► Assembly:

Lift the oval cap on the lid. Place the opening in the direction of the printing.

Firmly press on all the perimeter, until the lid clicks on the body



### ► Temporary closure

The container can be closed temporarily, pull down the cap until the small flap fits into the proper opening. The small flap remains in a horizontal position

### ► Permanent closure

From the temporary closure stage, firmly press with two fingers on the central "push" sign until it clicks.



At this stage the container can be reopened by raising the flap

At this stage, closure is permanent

## USE:

- During use, close the cap in temporary closure position
  - Close permanently once full
- 
- Large opening to ease the disposal of scalpels, test tubes, etc
  - Suitable to autoclave at 134° for 18 minutes
  - Once filled, the containers have to be disposed through incineration, releasing no toxic emissions

▶ **Needles removal:**

Use the “V” removal device to separate needles and syringes



▶ **Insulin pens removal:**

Use the “marguerite” device to unscrew the insulin pen



- ▶ The container is provided with handle



**ADVANTAGES:**

- ▶ Safe, easy to use and to handle
- ▶ Sharp and cutting objects resistant
- ▶ Provided with handle
- ▶ Lid provided with needles removal device
- ▶ Suitable to autoclave at 134° (compliant to ISO 23907-1:2019 even after sterilization)
- ▶ Maximum filling level indicated
- ▶ Irreversible permanent closure
- ▶ To be disposed through incineration, releasing no toxic emissions

**SAFETY INFORMATION:**

- ▶ Do not use the product before fitting the lid
- ▶ Do not attempt to rescue objects already in the container
- ▶ Respect the filling level indicated: 80% of real capacity
- ▶ Use the container only with the appropriate lid
- ▶ Do not try to reopen the container once permanently and irreversibly sealed
- ▶ Follow the instructions of use to ensure the safety of the container
- ▶ Container to dispose according to the existing law
- ▶ Use handles to move the container